

Emergency medical equipment should be checked daily to ensure its readiness for use. Using MyKitCheck removes the need for this check to include all equipment held. By using tamper evident digital safeguards users can be assured that all items within a trolley / bag are rescue ready. As a result a daily check of equipment can safely consist of checking the seal and external items such as defibrillator, suction and oxygen etc.

For more information watch the regular check video guide which can be found in the 'How to Guides' section on the home screen.



Open MyKitCheck

Access the MKC system using the method designated by your Trust e.g. desktop icon or go to app.mykitcheck.co.uk



Log In

Enter your login credentials. You must have an account registered before logging in using your Trust e-mail address.



Start a Check

Click Start a Check under Quick Actions
Select Regular Check
Enter Your Name if not already shown.



Confirm Tag/Seal

Enter the number from the tamper seal to confirm that it has not been changed. Be careful to enter correctly, if not recognised a full check will be required.



Check External Items

Check the external items as shown on the system, confirm quantities and expiry dates match those on packaging and update the system if required.



Submit Check

Once all items have been checked, leave any notes if needed and if any items require action. follow the on screen instructions.